
Instructional manual

Skin For Life


SFL-9001 9 IN 1 BEAUTY MACHINE

SFL-9001 9 IN 1 BEAUTY MACHINE

CONTENT


<u>Description of parts</u>	<u>1</u>
<u>Ozone steamer</u>	<u>4</u>
<u>High circular wave electric treatment apparatus</u>	<u>5</u>
<u>Facial toning</u>	<u>5</u>
<u>Galvanic instrument</u>	<u>6</u>
<u>Massage brush instrument</u>	<u>7</u>
<u>Vacuum</u>	<u>8</u>
<u>Electric Magnifying Lamp and Skin Analysis</u>	<u>9</u>
<u>Sterilized case</u>	<u>9</u>
<u>Specifications</u>	<u>10</u>
<u>Accessory list</u>	<u>10</u>

SFL-9001 9 IN 1 BEAUTY MACHINE


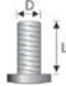
	D×L	D	L
	5/16in×5/8in	5/16in	5/8in
	15/64in×51/16in	15/64in	51/64in
	15/64in×51/32in	15/64in	25/64in

1. Install the 9 in 1 cabinet base① on the bottom board with screw 5/16in×5/8in.
2. Install the straight arm plate② on the 9 in 1 cabinet base① with screw 15/64in×51/64in.
3. Install the bent arm plate③ on the 9 in 1 cabinet base① with screw 15/64in×51/64in.
4. Install the spray machine unit④ into the bent arm plate opening③.
5. Install the Magnifying Lamp base⑤ into the straight arm plate opening②.
6. Install the accessory tray⑥ on the 9 in 1 cabinet base① with screw 15/64in×25/64in.
7. Install the netted tray⑦ inside the sterilizer container cabinet.
8. Install woods lamp fixture⑧ on back of standing cabinet with screw 15/64in×25/64in.


Description of parts

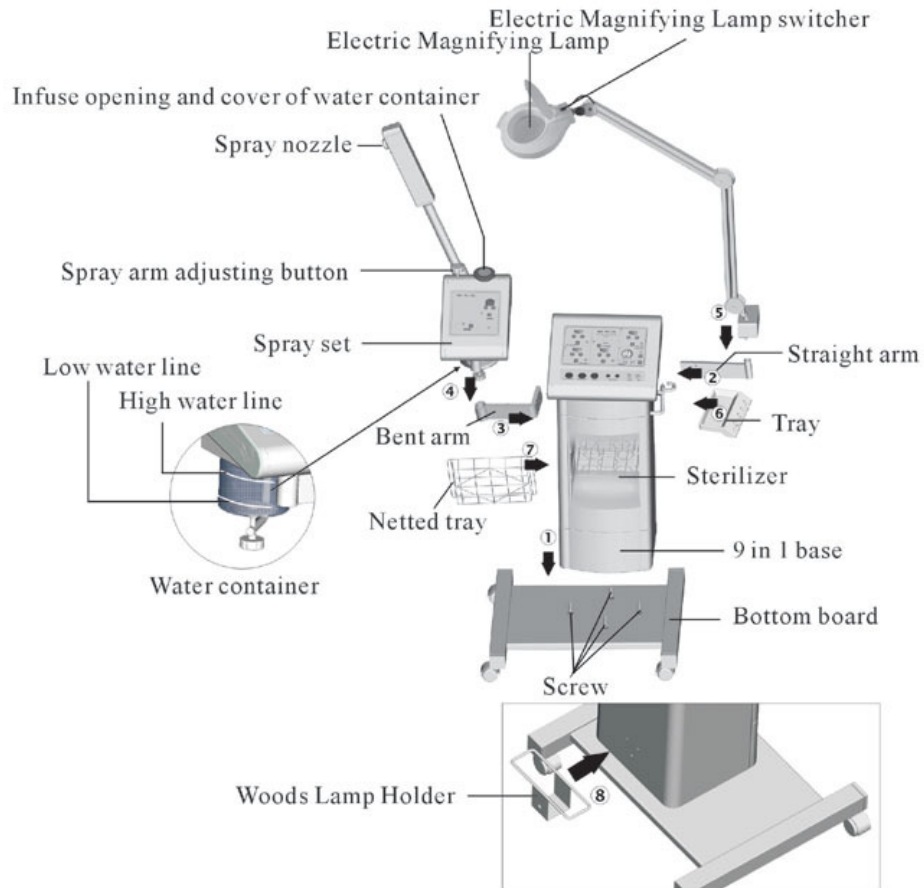

SFL-9001 9 IN 1 BEAUTY MACHINE

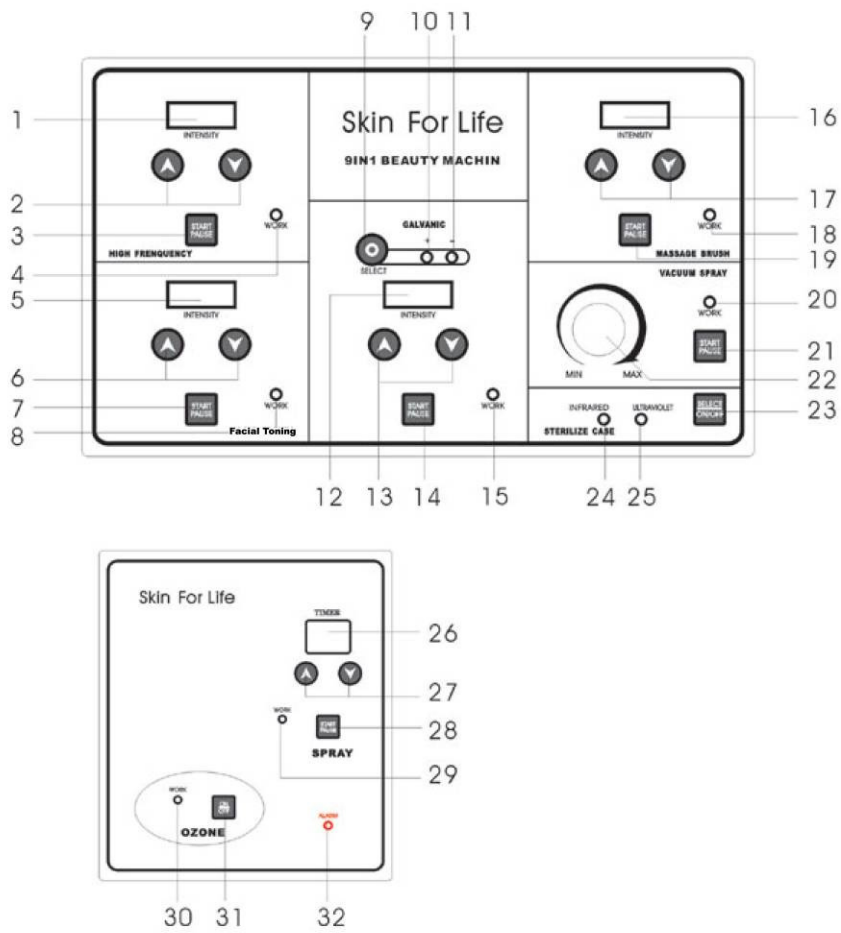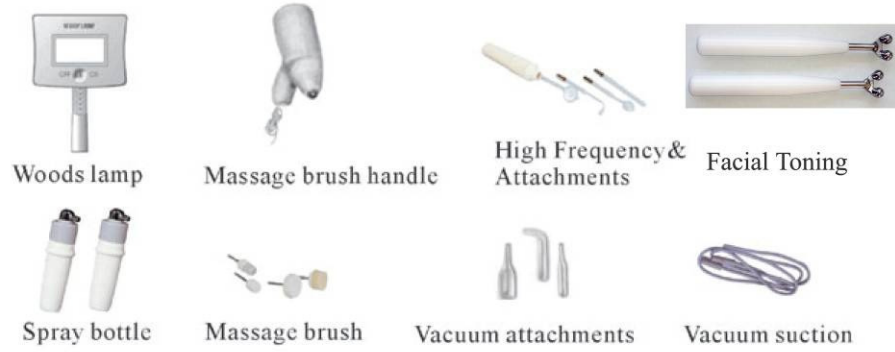

SFL-9001 9 IN 1 BEAUTY MACHINE

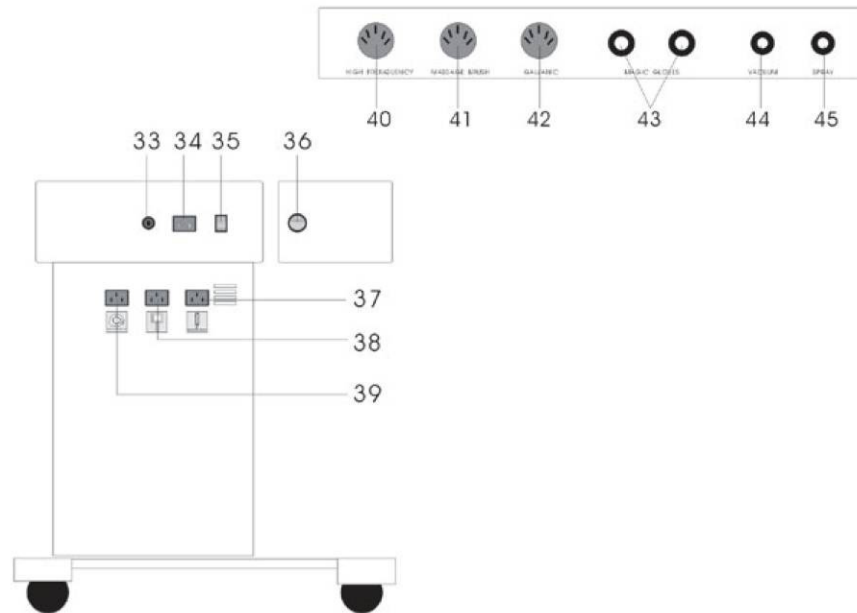

- | | |
|--|---------------------------------------|
| 1. High frequency intensity display | 24. Infrared rays indicator |
| 2. High frequency intensity adjust button | 25. Ultraviolet ray indicator |
| 3. High frequency start/pause button | 26. Sprayer time display |
| 4. High frequency indicator | 27. Sprayer time adjust button |
| 5. Facial toning intensity display | 28. Sprayer start/pause button |
| 6. Facial toning intensity adjust button | 29. Sprayer work indicator |
| 7. Facial toning start/pause button | 30. Ozone indicator |
| 8. Facial toning work indicator | 31. Ozone power switch |
| 9. Galvanic adjust key | 32. Low water indicator |
| 10. Infuse indicator | 33. Fuse |
| 11. Cleaning indicator | 34. Power |
| 12. Galvanic intensity display | 35. Main machine power switch |
| 13. Galvanic intensity adjust button | 36. Ozone power socket |
| 14. Galvanic start/pause button | 37. Ozone spray socket |
| 15. Galvanic indicator | 38. Skin Analysis socket |
| 16. Brush intensity display | 39. Electric Magnifying Lamp socket |
| 17. Brush intensity adjust button | 40. High frequency wave output socket |
| 18. Brush indicator | 41. Brush output socket |
| 19. Brush start/pause button | 42. Galvanic output socket |
| 20. Vacuum suction indicator | 43. Facial toning output socket |
| 21. Vacuum suction start/pause button | 44. Vacuum suction socket |
| 22. Vacuum suction intensity adjust button | 45. Spray socket |
| 23. Sterilization case power switch and select key | |

SFL-9001 9 IN 1 BEAUTY MACHINE

Ozone steamer

I Functions

Softens and hydrates the skin.

II Operations

1. Open water cover and fill container with distilled water only. Fill up to red line.
2. Press the ozone power switch 【36】. The Ozone indicator 【29】 light comes on, the time display is 30 minutes.
3. Press the sprayer start/pause key 【28】, right indicator 【29】 will flash. After 5-7 minutes a warm mist will begin. After steam appears, push the Ozone button 【31】 the Ozone will disinfect the water. This is done by the ultra violet light.
4. The programmed time expires, the steamer will switch off automatically.
5. When water level is low the indicator light 【32】 will flash and sound off ten times.

III Usage

Application and distances between the steamer nozzle and different types of skin can vary.

Skin texture	Distance	Use time
Neutral skin	25-30cm	Spray 3-5 minutes
Oily skin	20-25cm	Spray 5-8 minutes
Dry skin	30-35cm	Spray 3 minutes
Sensitive skin	35cm	Spray 5-8 minutes

CAUTION ON WATER LEVELS: VERY IMPORTANT!!!!

1. Add water to red line on glass container, do not let water level go lower than electric heating component.
2. Adjust distance between spray nozzle and face.
3. Maximum treatment should not be longer than 15 minutes according to skin condition.
4. Do not use Ozone spray on skin with small broken vessels.

ONLY USE DISTILLED WATER, WARRANTY IS VOID IF TAP WATER IS USED!

SFL-9001 9 IN 1 BEAUTY MACHINE

High circular wave electric treatment apparatus

I Functions

The High Frequency part of this machine works to kill bacteria that clogs pores and causes skin irritation. Antibacterial(great for acne, after facials and after waxing)

II Operation method

1. Insert the high frequency cord into the receptical[40].
2. Press the power switch [35], press the start/pause key [3], the indicator [4] light will illuminate, start treatment. The intensity output is "1", you should choose the intensity according to the skin condition.
3. After completing the procedure, please turn the high frequency button off, and clean accessories.

III Contrindications

Pregnancy
Seizures
Pacemaker
Heart conditions
Cancer

Facial Toning

I Functions

1. Uses a low frequency to enhance skin to muscle tonicity
2. Increases blood to lymph circulation for better cellular health

II Directions

1. Turn on the power switch[35], the intensity output[5] is "1".
2. Attach spheres to leads, then insert it into the output receptical[43].
3. Press the start/pause key, light will illuminate, intensity"1".
4. Press the intensity output adjust button[6], choose the intensity according to the treatment.

SFL-9001 9 IN 1 BEAUTY MACHINE

Galvanic Instrument

I Functions

The Galvanic Unit has two different functions. When set on the negative current(-) and using the circular(disk) electrode, the Galvanic Unit can be used for Deep Cleansing and Disincrustation. When it is set on positive current(+) it is used for penetration of water soluble products.

II Operating Procedure

1. Push down button[35], this turns on the power of the multifunction set, the indicator light will illuminate, the mode "+", the intensity output[12]is "1".
2. Push down button[14], push to indicate(-) or (+). Light will illuminate depending on the mode.
3. Push button[9] to select the (+) or (-), so a(5) would be discincrustation.
4. Client will hold ground rod. This should be wrapped with a moist cloth.
5. Press button[13] to adjust the output level to the tolerance level of client.

III Usage

Contra-indication

Pacemaker, Pregnant, Heart Disease and Seizures

Deep Cleanse-Disincrustation

- Cleanse face
- Use alkaline based solution to soak facial gauze or use your own product.
- (Baking sode, mix one level teaspoon with one cup of water)
- Squeeze facial gauze out and place on the face(Make sure the gauze stays moist)
- Attach (disk, roller or balls)
- The ground (solid metal) should be wrapped with a wet peice of gauze and have the client hold it in their hand.
- Turn machine on and choose the(-) negative settings.
- Adjust intensity to clients comfort.
- Work 5-10 minutes, working from the center of the face out to hairline.
- Client may taste experience a metallic taste.
- Procedure opens pores, softens sebum, dissolves grease and softens blackheads.
- Excellent treatment before extractions or microdermabrasion.

SFL-9001 9 IN 1 BEAUTY MACHINE

Product Penetration

- After facial or micro current.
- Apply thin layer of product (any water soluble product) keep moist.
- Use any attachment (balls, roller)
- The ground should be wrapped with a dampened gauze and have the client hold it in their hand.
- Choose the (+) positive setting.
- Adjust for optimum client comfort.
- Treat 5-10 minutes.
- Clients may experience a metallic taste.
- Slight tingle, begins to feel warm after a few minutes.
- Procedure infuses products into the skin, calms down inflamed tissue, slows down oil production to sedate and re close the pores.

Massage brush instrument

I Functions

1. To deep cleanse.
2. To improve blood circulation and smooth fine line and wrinkles, also acts as a massage.

II Operational methods

1. Push down [35] button to turn on multifunction set, the intensity [16] is "1".
2. Attach brush to lead, then insert it into the output socket [41].
3. Push down start/pause key [19] to turn on the massage brush, light indicator is on, brush treatment.
4. Turn the intensity output button [17] to adjust the speed.
5. After use, turn down its speed, then push [19] to switch off power.

III Usage

1. Large brush is applied on forehead, cheek, low jaw, small brush is applied on nose, nose ridge.
2. After 2-3 minutes, switch off and take off the brush, cleanse the skin.
3. Install the massage head and apply massage cream on skin, adjust speed level while turning on machine.
4. While massaging, press down slightly.
5. After massaging for five minutes, turn off the power.

IV Attention

- 1) Cleanse gentle and softly.
- 2) Do not use on acne infected, broken, tender or sensitive skin.

SFL-9001 9 IN 1 BEAUTY MACHINE

Vacuum

I Functions

1. Lymphatic treatment with vacuum attachments, follow face chart for lymphatic, working from the center of the face to the hairline. Then flush from forehead with glass attachment of choice, follow down the hairline to the neck.
2. Spray cooling mist to calm the skin.
3. Choose the attachment.

II Directions

1. Insert the glass attachment into the tubing, insert the other end into receptical[44] or [45].
2. Turn on the power switch[35], press the start/pause button, the indicator[20] illuminated.
3. Adjust the intensity button to the right, then adjust the output level according to skin type.
4. Push[21] to switch off, then clean accessories.

III Usage

I. Vacuum suction

- 1) Insert the glass attachment into tube, then turn it on, adjust intensity level.

2) There are three types of suction methods

- ① Interruptive suction: suitable for larger areas.

Use thumb and index finger, hold glass tube during operation, push the tube opening against the skin, with middle finger tap frequently on ventilation hole of the glass tube, to form suction effect, the move over the skin. Move fast, suction is quick and rhythmic.

- ② Consecutive suction: suitable for oily, thick skin.

Use thumb and index finger hold glass tube during operation, pose the tube opening against skin, with middle finger close the ventilation hole of glass tube along with the movement of suction tube, release till it reaches the verge. It induces more powerful suction.

- ③ Power suction: suitable for applying on oily location

Use thumb and index finger, hold glass tube during operation. Push the tube opening against skin. Close ventilation hole with finger, the glide across the skin, quick and rhythmic.

SFL-9001 9 IN 1 BEAUTY MACHINE

2. Cool spray

1. Insert the spray bottle into soft tube. With middle finger and thumb, hold the base of the bottle. With index finger press ventilation hole on top of spray bottle.
2. Beautician hold the bottle spray from forehead to cheek, avoid water enter nostrils.

IV Attentions

1. Should not apply on same location for a long time, move fast, avoiding local blood congestion.
2. Adjust suction power according to skin texture of clients.
3. Co-use with O₃ spray will be more effective.
4. Do not use vacuum on skin around the eyes.

Electric Magnifying Lamp and Skin Analysis

Directions

1. After skin is cleaned, cover the eyes of clients with wet cotton patch or ask them to close eyes.
2. Insert Magnifying Lamp or Skin Analysis plug into the correspondent output socket[38] [39], then press the start/pause key.

Sterilized case

operating Procedure

1. Press the power switch[23], the indicator[24] light will illuminate and now is at infrared setting.
2. Press the power switch select key[23], the indicator light will go out, the UV indicator[25] will now sterilize your attachments.
3. Press the select button[23] a third time both the infrared(heat) and the UV light (sterilizer) are both on.
4. Press the [23] button a fourth time and power is off.

SFL-9001 9 IN 1 BEAUTY MACHINE

Specifications

1. Power: AC110V/60Hz

Accessory list

- | | |
|-----------------------------|---------------------|
| 1. Spray main set | 1piece |
| 2. Multifunction set | 1piece (with stand) |
| 3. Magnifying Lamp | 1piece |
| 4. Skin Analysis woods lamp | 1piece |
| 5. Container cover | 1piece |
| 6. Glass water container | 1piece |
| 7. Galvanic ground | 1set |
| 8. Massage handle | 1piece |
| 9. Massage brush complex | 1set |
| 10. Galvanic attachment | 1piece |
| 11. Galvanic attachment | 4pieces |
| 12. Spray bottle | 2pieces |
| 13. Suction and spray tube | 2pieces |
| 14. MicroCurrent spheres | 2 pieces |
| 15. Glass attachments | 3pieces |
| 16. Power cord | 1piece |
| 17. Manual | 1book |
| 18. Fuse | 2granules |

Skin For Life

SCRUBBER&GALVANIC SFL-201

