

Galvanic Treatment System Instructions

CONTRAINDICATIONS:

Pregnancy, seizures, pacemakers, heart conditions and cancer. Your client should be questioned for any of these conditions and treatment refused if one or more such conditions are present. If your client is not sure, have the client contact their physician.

Description:

The High Frequency part of this machine works to kill bacteria that clogs pores and creates skin irritation. Antibacterial (great for acne, after facials and after waxing). The Galvanic Unit has two different functions. When set on the negative current (-) and using the ball/roller electrode, the Galvanic Unit can be used for a Deep Cleanse - Disincrustation. And when set on a positive current (+) and using the ball/roller electrode, the Galvanic can then be used for product penetration (water-soluble products).

Galvanic Function

A) Deep Cleanse

- Press button to turn galvanic function on. Adjust up and down for intensity.
- Insert long metal rod electrode (a.k.a. the Ground Electrode).
- Cleanse the patient's face
- Take gauze and soak it in an alkaline based solution (one tsp. baking soda w/ one cup water) or your own product
- Place wet gauze onto the patient's face
- Wrap the Ground Electrode in another damp gauze (dampen with water) and have the patient hold it.
- Screw attachment into the handle and tighten, making sure it is in securely.
- Press start/pause button for the Deep Cleanse, you need a negative current (-).
- This attachment is used to soften sebum, dissolve grease, soften blackheads and open pores.
- Apply ball/roller electrode to different areas of the face. Make sure to always move down and away from the forehead, eyes, nose and chin towards the ears, lower jaw and lower neck because this allows for the best facial lymphatic drainage.
- Patient may feel a slight tingle at first but after a few minutes it will begin to feel warm. Also they may taste metal.
- Adjust the current to the patient comfort level by pushing the arrow up or down (intensity).
- This procedure can be performed before microdermabrasion.

B) Product Penetration

- Use the roller and ball for this treatment. Start at the center of the face and roll down and out to the hairline. Just follow face chart.
- After facial make sure to keep face moist and paint skin with product (water-soluble).
- Again have the patient hold the Ground electrode wrapped in wet gauze.
- Press button so there is a positive current flow (+).
- Starting in the center of the face, work out, working in the same direction as with Deep Cleanse (see face chart).
- Patient may feel a slight tingle.
- What this part of the galvanic function is doing is pushing the product into the patients face. It should calm inflamed tissue, and slow down oil production.
- At the end of treatment, making sure you are still using a positive current.
- Use the roller electrode to help sedate and reclose tighten pores and lymphatic drainage.
- Again adjust intensity to clients/patients comfort level.