

Magic Gloves Instructions

- Attach the red lead and black lead to front of machine.
- Place gloves into a bowl of water for few minutes until completely absorbed.
- Place plastic gloves on hands, snap on the black and red lead to magic gloves.
- Push start/pause, and then increase intensity per comfort level of client.
- As the gloves start to dry, spray them with water.
- Cleanse the face, apply (water soluble) product on face and neck.
- Starting at the neck and work up to the face.
- See chart.
- Replace plastic gloves(made of an insulated plastic) after every treatment. - Re-order the gray magic gloves after 200 treatments.
- This will help to stimulate collagen and firm and tone the skin.