

Hydraulic bed instructions (FBD210)

Skin for Life

1-866-312-7546 (SKIN)

<http://www.skinforlife.com>

Click [here](#) for other product instructions.

Step 1:



Remove base from package and set in position. Be sure to place the foot pump in a convenient location. Remove the bolt that holds the block of wood in place. Keep the bolt, it will be needed later.

Step 2:



Remove the padded portion of the chair, notice the notch in the base.

Step 3:



Place padded on the base , make sure the notch and the pin are in alignment.

Step 4:



Use bolt saved from step 1 to secure top to base. Access is possible from the headrest side of the chair.

Step 5:



Once the two halves are together, place the 2 long steel rod on the "head" side of the bed and the shorter rods at the "foot" portion of the bed.

Step 6:



Remove 1 Phillips screw from the end of each of the rods.
Do not discard the screws.

Step 7:



Take rods for the headrest side of bed and slide through
chrome locking plates hanging under the chair. Replace
the screws in each rod.

Step 8:



Next place bolt through the bracket and the rod on back of bed.

Step 9:



Tighten the bolt so it is SNUG not TIGHT. Follow the same procedure for the "foot" side of bed.

Step 10:



Remove armrests from packaging and place pins in holes as indicated.

Step 11:



To raise bed push down on foot pump in short strokes. To lower bed push all the way down on the foot pump and hold it down.

Step 12:



To lock bed and prevent unwanted twisting raise the foot pump with your toe. This will minimize ,not eliminate the rotation of the bed. Do NOT swivel the bed when patient is lying or sitting on the bed. This may cause the bed to tip.

Any further questions?
866-312-7546
